

"I work out, but it still doesn't tighten up the areas that I am concerned about. With VelaShape, I am able to target these areas. I noticed that the flank area went down significantly. I was really impressed! The treatment feels like a warm massage, so there is no pain. I absolutely believe it has been worth the money."

M.C., USA

Syneron & Candela

two extraordinary brands, one global leader.

Syneron has a remarkable ability to innovate the newest technologies in aesthetic medicine.

Candela brings four decades of industry-leading scientific expertise.

Together, these two brands offer "gold standard" technologies for advanced aesthetic solutions.



Science. Results. Trust.



Science. Results. Trust.

VelaShape III®

body contouring & cellulite treatment

infois@syneron.com | www.syneron-candela.com

Syneron

TM&©2013 Syneron Medical Ltd.

CANDELA

* This brochure is not intended for use in the U.S. market.
©2013. All rights reserved. VelaShape, Syneron, the Syneron logo and elos are trademarks of Syneron Medical Ltd. and may be registered in certain jurisdictions. elos (electro-optical synergy) is a proprietary technology of Syneron Medical. Candela is a registered trademark of the Candela Corporation. PB83691EN

CE 0344

Syneron

CANDELA


VelaShape III body contouring & cellulite treatment



Treat Yourself To a Sexier,
Smoother Figure - Visible Results
in as Little as 1 Treatment

VelaShape III™ contours, shapes and slims the body by improving cellulite and reducing circumference in as few as 1 treatment session. VelaShape provides dramatic results without downtime or significant discomfort.

The secret behind VelaShape is that it is the first and only technology that combines elōs with vacuum and tissue manipulation to reduce the volume and appearance of fat tissues.

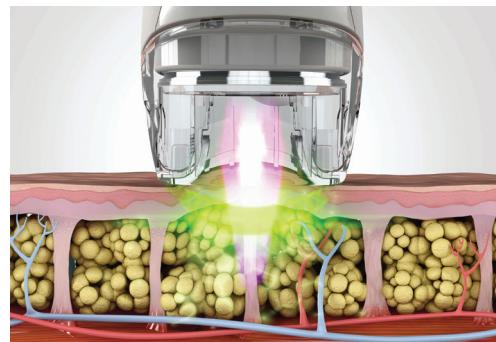

VelaShape Technology: A combination of elōs-infrared + bi-polar radiofrequency energy-and vacuum for a reduction in skin laxity, volume and an overall improvement in skin texture

How quickly will I notice a change?

Gradual improvement of the treated area can be seen following the first treatment – with the skin surface of the treated area feeling smoother and firmer.

How many centimeters can I reduce from my circumference?

In clinical studies, patients report an average reduction of 1 - 3 centimeters post 1 treatment. A recent study showed 100% of patients had at least 1.5 cm reduction.

How does VelaShape III work?

The elōs technology combination of infrared light, coupled radio frequency energy and vacuum causes deep heating of the fat cells, their surrounding connective tissue and the underlying dermal collagen fibers. This type of efficient heating and vacuum stimulate the growth of new and better collagen and elastin which results in localized reduction in skin laxity, body volume and an overall improvement in skin structure and texture.

Is treatment safe?

Treatment is safe and effective for all skin types and colors. There are no reported short or long-term health effects.

Does it hurt?

Most patients find VelaShape comfortable – like a warm deep tissue massage. The treatment is designed to accommodate your sensitivity and comfort level. It is normal to experience a warm sensation for a few hours post treatment. Your skin may also appear red for several hours.

Clinical Results



Before



4 Weeks Post 1 Treatment
Abdomen: -4 cm



Before



Post 1 Treatment
Abdomen: -2.5 cm



Before



Post 2 Treatments
Abdomen: -3 cm

Photos: Ruthie Amir, MD